

9L / 13L / 24L / 30L

Assemble with fill line and label to the front

This ensures you can always see when the container is full.



It just takes four clicks

4 audible clicks confirm the lid and base are assembled correctly and securely.



Fill in the label before and after use

This ensures a permanent audit trail.



Never fill above the line

Always replace your sharps container when full to avoid needle stick injuries.



Temporary closure

The temporary closure feature should always be engaged when the container is not in use.

To engage the temporary closure feature, pull the flap up using the central tab, rotating the flap until it covers the aperture.



Final closure

The final closure feature should always be engaged when the container is filled to the fill line.

To engage, place the container on a suitable surface. Pull the flap using the central tab, rotate it to cover the aperture then firmly apply pressure along the front of the flap.



FOR MORE INFORMATION:

✉ sharpsafesales@vernagroup.com

🌐 vernacare.com 📞 +44 01495 235800