

# sharpsafe®

## The all new 5 & 7L Sharps Containers

With DUAL PROTECT **PRO**™

IFU – ENGLAND & WALES



**1 To assemble your container**  
Face the fill line and label forward.



**2 It just takes four clicks**  
Four audible clicks on each corner of the lid confirm that the lid and base are assembled correctly and securely.



**3 Fill in the label before use**  
This ensures a permanent audit trail.




**4 Never fill above the line**  
Always replace your sharps container when full to avoid the risk of needlestick injuries.



**5 Temporary Closure**  
To engage, gently press the lid down until the aperture is covered. To release, pull back on the tab and lift the lid until it engages in an open position.



**6 Final Closure**  
Once full, permanently close the container. With the container in temporary closure, press the lid down firmly until an audible click is heard.

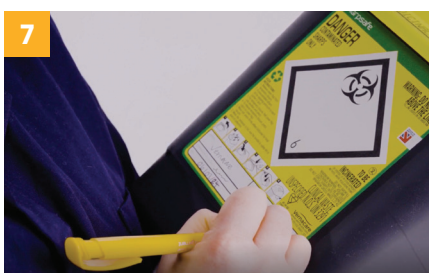

**7 Final Disposal**  
Fill in the disposal label and follow local disposal guidance for sharps containers.

Pocket Small Mid Large Specialist Accessories

**Safer today for a sustainable tomorrow.**

1 Western Avenue, Matrix Park, Buckshaw Village, Chorley, PR7 7NB, UK  
+44 (0)1204 529494 hello@vernacare.com vernacare.com

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. © Copyright Vernacare International Limited 2024. All rights reserved. CWM015 REV00



**Vernacare**  
clinical waste  
management